

step-by-step instructions

Opening protective outerwrap



step 1: Tear down vertically from right side of the serrated edge of the bag (edge opposite the access ports).



step 2: Tear down vertically from left side of the serrated edge of the bag (edge opposite the access ports).



step 3: Lift the plastic outerwrap, beginning at the bottom of the bag and pulling upward, to release the vacuum seal.



step 4: Pull bag out of the overwrap.

Opening and reconstituting 2-compartment dialysis solution bags



step 1: After removing the protective outerwrap, lay the bag down on a flat surface.



step 2: Squeeze bag sides of small compartment inward to open or “pop” the peel seam.



step 3: Press on the large chamber, opening the entire peel seam.



step 4: Mix the contents by rotating the bag at least 5 times back and forth.

This document is not intended to replace the NxStage PureFlow B Solutions Instructions for Use and does not include all of the information necessary to use the product safely and effectively. Providers should review and refer to the PureFlow B Solutions Instructions for Use for complete information including all warnings and precautions.